

# Diversey

# Oxivir® Five 16 Concentrate

One-Step Disinfectant Cleaner A one-step disinfectant cleaner based on proprietary hydrogen peroxide (AHP®) technology.

# Virucidal, Bactericidal, Fungicidal and Mildewcidal

- Disinfects in just five minutes
- Effective against Human Immunodeficiency Virus HIV-1 (AIDS Virus), Hepatitis B and Hepatitis C Virus (HBV and HCV), Pseudomonas aeruginosa, Staphylococcus aureus, Methicillin resistant Staphylococcus aureus (MRSA), Salmonella enterica, Avian Influenza A and VRE
- Kills Norovirus and Canine Parvovirus

### Compatible With Hard Surfaces\*

- Active breaks down into water and oxygen by-products
- No rinsing required
- Delivers fast and effective cleaning







# Oxivir® Five 16 Concentrate

# One-Step Disinfectant Cleaner



#### **Use Directions**

- Let solution remain on surface for a minimum of 5 minutes.
- Wipe and allow to air dry.
- Prepare a fresh solution if the solution becomes visibly dirty or diluted.

#### Dilutable Disinfectant Cleaner

- 1:16 dilution One-step hospital grade disinfectant cleaner in 5 minutes.
- 1:64 dilution Virucidal in 5 minutes, meets bloodborne pathogen standards.
- 1:128 dilution Non-food contact sanitizer in 3 minutes.
- 1:256 dilution Floor and general cleaner.

## **Product Specifications**

Description	
Color/Form	Colorless, liquid
Flash Point	> 200° F (> 93.4° C)
EPA Reg. No.	70627-58
рН	1.0 (Concentrate) 1.9 (Use Dilution 1:16)
Scent	Characteristic
Shelf Life	3 years
Solubility	Complete
Specific Gravity	1.036

#### SmartDose<sup>™</sup> Icons



Spray Bottle 1:16



Mop Bucket 1:256

#### **Available Items**

Product Code	Description/Package Size	Dilution
4963314	4 x 1 gallon / 3.78 L Containers	1:16, 1:64, 1:128, 1:256
4963331	2 x 2.5 L / 84.5 oz. J-Fill® Bottles	1:16
4963357	2 x 1.5 L / 1.59 Qt. RTD° Units (Fill)	1:16, 1:64, 1:128, 1:256
5019296	2 x 1.4 L / 1.48 Qt. SmartDose™ Bottles	1:16 (Spray Bottle), 1:256 (Mop Bucket)
5271361	2 x 1.5 gallon / 5.67 L Command Center® Units	1:16, 1:64, 1:128, 1:256

### Safety Reminder

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use, and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDSs are available on-line at www.sealedair.com or by calling 888 352 2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, http://naextranet.diversey.com/dot/, for up to date shipping information.

AHP and Design® are registered trademarks of Virox Technologies, Inc. used under license.